

ANALYSING A CONTROL

PREPARE FOR TESTING

Before starting up – please note!

- The Test Cartridge and the control material must reach room temperature (15-30°C, 59-86°F) before use. Leave the unopened foil pouch at room temperature for at least 15 minutes.
- Be sure that the Analyzer is switched on and ready for use.
- Mix the control material thoroughly before use.
- The Test Cartridge has to be analysed within 1 minute from capillary filling.



Consult the Afinion™ HbA1c and HbA1c Control Package Inserts for further information on sample material and handling.




- 1** Open the foil pouch just before use. Use the handle for correct grip.

COLLECT SAMPLE



- 2** Pull the sampling device straight up from the Test Cartridge.

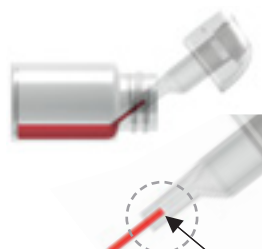

- 3** Fill the capillary completely with control material. Do not wipe off the capillary.



- 4** Immediately replace the sampling device into the Test Cartridge.

START TESTING



- 5** Touch to enter control mode. The lid opens automatically.



- 6** Insert the Test Cartridge with the barcode label facing left.



- 7** Close the lid manually.

RECORD RESULT



- 8** Touch and enter control ID during processing. Touch to confirm.



- 9** Record the result, then touch to accept. The lid opens automatically.



- 10** Remove and discard the used Test Cartridge. Close the lid manually.

ANALYSING A PATIENT SAMPLE

PREPARE FOR TESTING

Before starting up – please note!

- The Test Cartridge must reach room temperature (15-30°C, 59-86°F) before use. Leave the unopened foil pouch at room temperature for at least 15 minutes.
- Be sure that the Analyzer is switched on and ready for use.
- The Test Cartridge has to be analysed within 1 minute from capillary filling.



Consult the Afinion™ HbA1c Package for further information on sample material and handling.



1

Open the foil pouch just before use. Use the handle for correct grip.

COLLECT SAMPLE




2

Pull the sampling device straight up from the Test Cartridge.



3

Fill the capillary completely with patient sample. Do not wipe off the capillary.

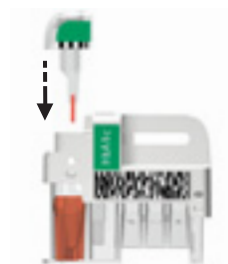


4

Immediately replace the sampling device into the Test Cartridge.

START TESTING



5

Touch  to enter patient sample mode. The lid opens automatically.



6

Insert the Test Cartridge with the barcode label facing left.





7

Close the lid manually.

RECORD RESULT




8

Touch  and enter patient ID during processing. Touch  to confirm.



9

Record the result, then touch  to accept. The lid opens automatically.



10

Remove and discard the used Test Cartridge. Close the lid manually.